

Masimo SafetyNet™

Setup Instructions

Warning: Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.

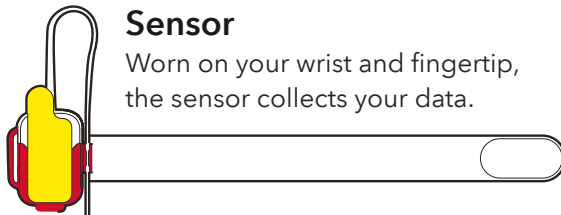
GET TO KNOW YOUR SYSTEM



Masimo SafetyNet App

Masimo SafetyNet displays Oxygen Saturation, Pulse Rate, Respiration Rate and other data from Radius PPG. It also allows data updates to your care provider.

Radius PPG



Sensor

Worn on your wrist and fingertip, the sensor collects your data.



Chip

Snapped into the wrist portion of the sensor, the chip sends your data to the Masimo SafetyNet App.

Getting Started

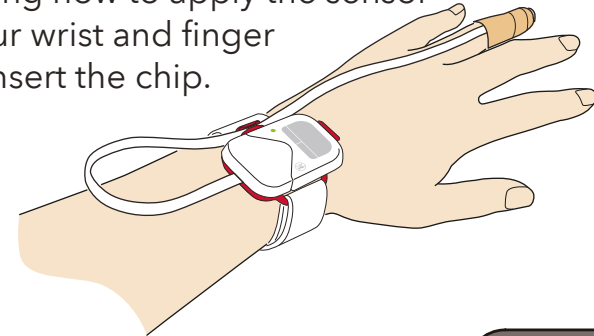
1 Accept Masimo SafetyNet Invitation

The invite will come to your email. After you accept, download the Masimo SafetyNet App and follow setup instructions.



2 Apply Sensor

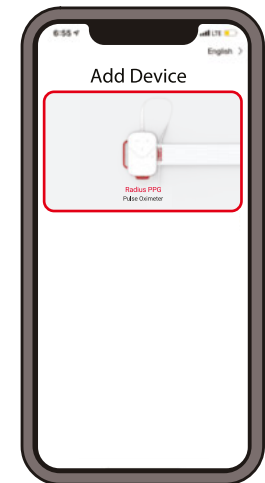
See instructions on the back showing how to apply the sensor to your wrist and finger and insert the chip.



3 Pair Radius PPG

Under Masimo SafetyNet App **Settings**, enter the **Add Device** menu, and follow instructions for pairing the Radius PPG to your phone.

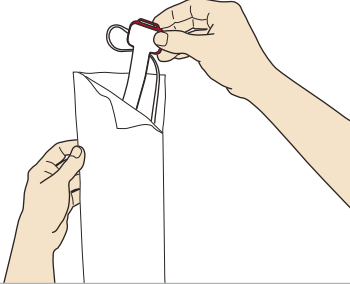
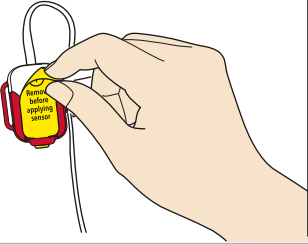
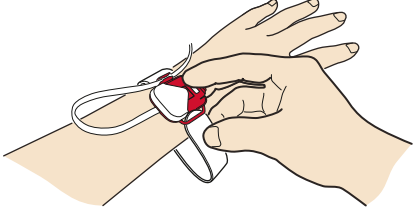
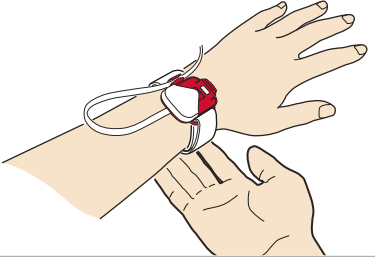
For more info go to:
masimo.com/masimo-safetynet



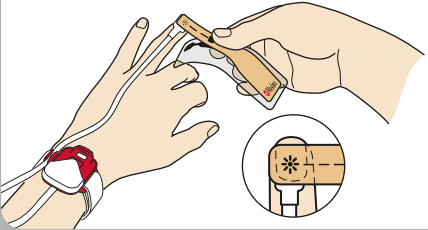
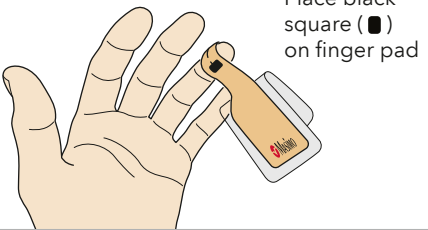
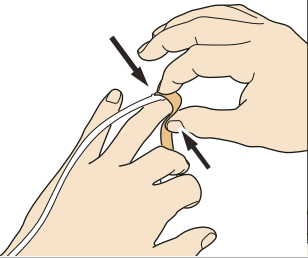
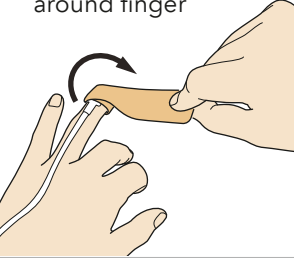
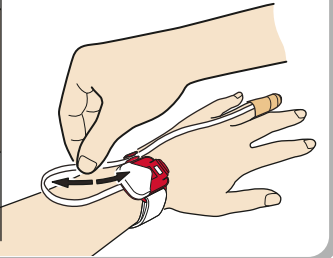
see back for sensor application →

How to Apply your Sensor

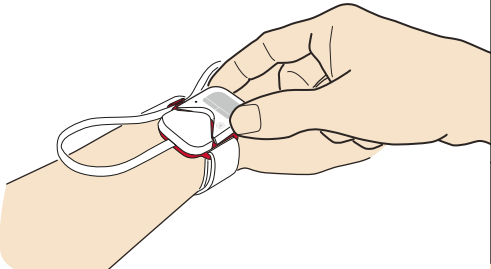
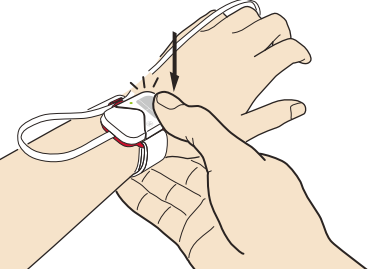
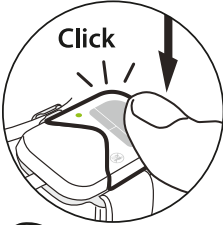

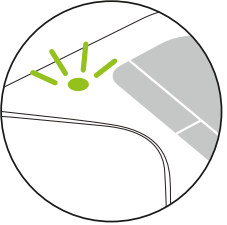

2.1 Sensor Set Up

<p>1. Remove sensor from pouch</p> 	<p>2. Peel off yellow label to activate battery</p> 	<p>3. Place sensor on wrist as shown and carefully thread the tip of the strap through the open red loop</p> 	<p>4. Wrap the loose end of the strap around your wrist and press</p> 
--	---	--	---

2.2 Apply Finger Sensor

<p>1. Remove part of release liner and place star (✱) on top of fingernail of ring finger</p> 	<p>2. Wrap halfway around finger. Make sure the star and square are lined up directly across from each other</p> <p>Place black square (■) on finger pad</p> 	<p>3. Squeeze top and bottom of wrap to secure</p> 	<p>4. Remove release liner and continue to wrap remainder around finger</p> 	<p>5. Adjust cable to comfortable length</p> 
--	--	---	--	---

2.3 Insert Chip

<p>1. Insert the chip into the open compartment on the sensor</p> 	<p>2. Press down on the chip to secure</p> 	<p>Click</p>  <p>Listen for click to secure.</p> 	 <p>Look for light to confirm connection.</p> 
---	---	---	--