

## **RAD-5V PULSE OXIMETER**

- The ON/OFF switch is at the bottom left of the front of the machine. When tuning off RAD-5 press ON/OFF button for 2 seconds
- ☀ Pressing this symbol brightens and darkens the screen. The brighter the screen the more power is used
- 🔊 Pressing this symbol increases and lessens the sound of the Heart Rate
- The BELL with a line through it silences the alarm. The machine only alarms when you take the sensor off.
- **To Change Batteries**
  - Remove Cable from RAD-5V
  - Remove Protective Cover from RAD-5
  - Remove Battery cover at the back of RAD-5V
  - Remove the batteries and replace