



BabyControl Apnoea Monitor

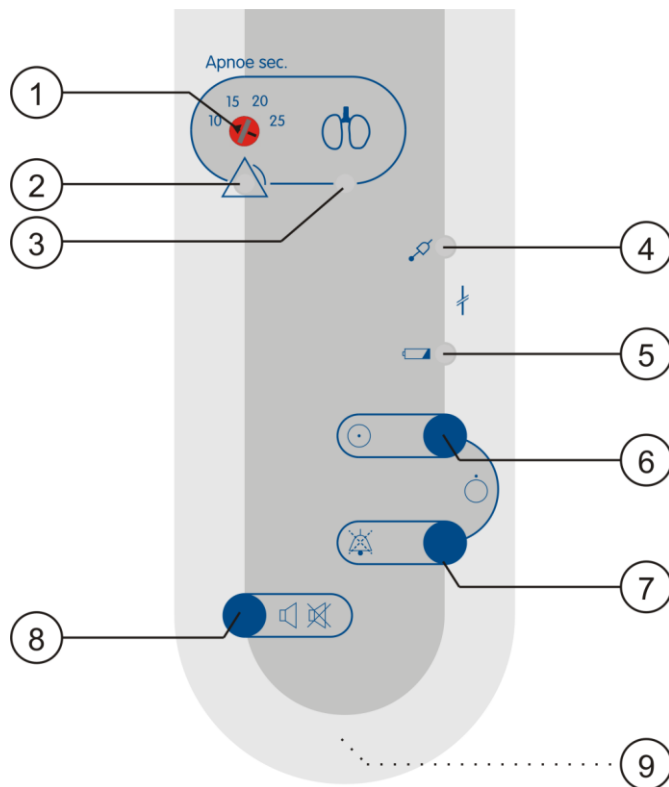


1		Switch for respiration alarm delay
2		Alarm signal: respiration
3		Function signal: respiration (green)
4		Respiration sensor fault (yellow)
5		Alarm signal: battery discharged
6		ON button / to switch on BABYCONTROL
7		Reset button for Alarm
8		Button for respiration monitoring tone: On/Off
9		Battery compartment (back of monitor)

Brief Instructions: SISS BabyControl Apnoea Monitor


The following notes on first use will help you to quickly get used to operating the monitor.

1. Apply the blue respiration sensor as in Fig. 3. Secure the respiration sensor to the child's body with the Leukosilk or Leukopor sticking plaster supplied. One horizontal strip across the sensor will suffice. The respiration sensor must not be kinked.
2. Connect the respiration sensor to the 3-pole socket of the monitor
3. Check the alarm limit set on the monitor.
4. Switch on the monitor by pressing the button  (6) and keeping it pressed for about 1 second.
5. The monitor then carries out a self-test and, after emitting an acknowledge tone, starts reliable monitoring.


Applying the respiration sensor

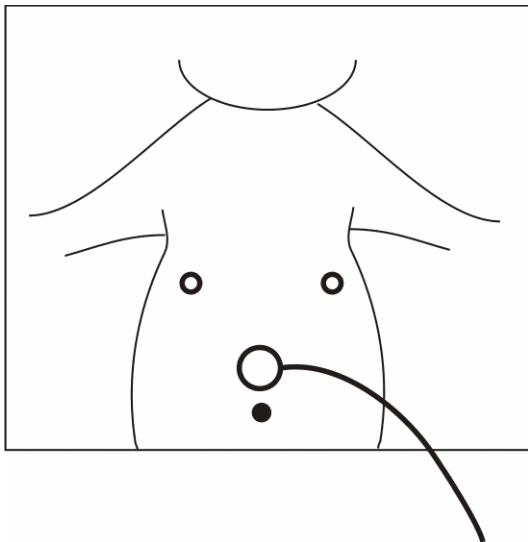


Recommended location of the respiration sensor

Switching on: Press 

Switching off: Press  and  together for 1 sec.

Press  to silence the alarm.

Press   to switch the respiration tone on and off.