


## Quick sheet For Downloading Data from SISS Baby Control

1. Plug USB Transponder – Pad in USB port on computer.
2. Place SSIS Baby control on transponder pad.
3. From the **START** menu on your computer open **SSIS Start**.
4. Select **start** on right hand side under **Transponder Pad**.
5. Left Click on **Start** on the top left of screen.
6. Left click on **Read device**.
7. Check that **Date and Time** are correct and then left click on **connect**.
8. Press on **Sound and Reset Keys (Keys 7 & 8)** on Baby control simultaneously.
- 9.

*Control elements*

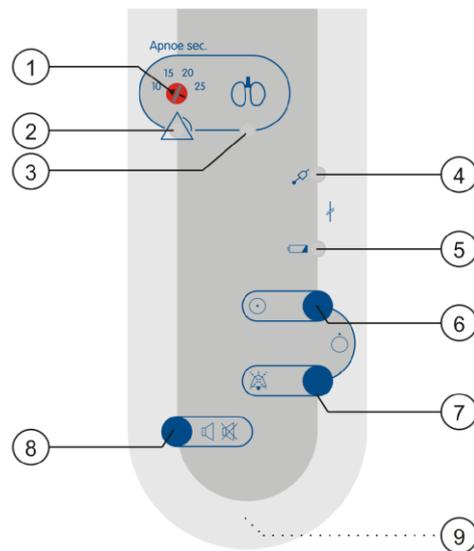

10. **Sound and Rest Keys** will flash.
11. Data will start transmitting.
12. Insert patient details or select existing patient.
13. Save data.