

sat 801

Pulse Oximetry System

Technical Information and Service Manual



CONTENTS

1.		PRELIMINARY REMARK	4
2.		FUNCTION DESCRIPTION	•5
	2.1.	INTENDED USE	5
	2.2.	CONTRAINDICATION	5
	2.3.	PRODUCT DESCRIPTION	5
	2.4.	THEORY OF OPERATION	6
3.		TECHNICAL DESCRIPTION OF THE SAT 801	7
4.		ASSEMBLY	7
•	4.1.	DISPLAYS AND CONTROLS	.7
	4.1.1	. Front	7
	4.1.2	. Back	7
	4.2.	CONNECTIONS ON THE UNIT	8
	4.2.1	. Block diagram	.8
	4.3.	POWER SUPPLY	.9
	4.4.	ON/OFF BUTTON1	0
	4.5.	MASIMO BOARD	0
	4.6.	MICROPROCESSOR AND PERIPHERALS1	0
	4.7.	SELF-DIAGNOSTIC CAPABILITY OF THE SAT 801	11
	4.7.1	. Watchdog and undervoltage reset	11
	4.7.2	ROM check	11
	4.7.3	. RAM check	11
	4.7.4	L. Clock module check	11
	4.7.5	. Communication check with the Masimo board	11
	4.7.6	5. Operating voltage check 1	2
	4.7.7	Loudspeaker check 1	2
	4.7.8	8. Stack monitoring1	2
	4.7.9	Detection of an uncontrolled reset 1	2
5٠		MAINTENANCE PROCEDURES1	4
	5.1.	PRELIMINARY REMARK 1	4
	5.2.	BATTERY REPLACEMENT 1	4
6.		POSSIBLE ERRORS AND THEIR CORRECTION 1	15
	6.1.	ERROR: SYSTEM ALARMS	15
	6.1.1	. Hardware-related system alarms	15
	6.2.	ERROR: GREEN AC POWER LEDS DO NOT COME ON	15
	6.3.	ERROR: SAT 801 CANNOT BE SWITCHED ON IN BATTERY MODE	15
	6.4.	ERROR: SAT 801 WILL NOT SWITCH ON IN MAINS MODE	15
	6.5.	ERROR: YELLOW AND/OR RED ALARM LEDS DO NOT COME ON	15
	6.6.	ERROR: FAULTY LOUDSPEAKER 1	6
	6.7.	ERROR: DISPLAY VALUES NOT PLAUSIBLE AND/OR FAULTY 1	6
7.		SAT 801 CHECK ROUTINE1	7
	7.1.	PRELIMINARY REMARK 1	17
	7.2.	POWER SUPPLY CHECK 1	17
	7.3.	DEVICE SELF-TEST AND BRIEF CHECK 1	17
	7.4.	ALARM LIMITS CHECK 1	8

7.5.	SENSOR ALARMS CHECK	
7.6.	ACOUSTIC SIGNALS CHECK	
7.7.	DEVICE VALIDATION	
7.8.	BRIEF CHECK	
8.	SERVICE PROGRAM SERVICE801	
8.1.	PRELIMINARY REMARKS	
8.2.	GENERAL INFORMATION ABOUT THE SERVICE801	
8.3.	PERFORMING A PARAMETER UPDATE	
A.	PERFORMING A SOFTWARE UPDATE	
9.	DATA DOWNLOAD PROGRAM DOWNLOAD801	23
9.1.	PRELIMINARY REMARK	
9.2.	GENERAL INFORMATION ABOUT DOWNLOAD801	
- 9.3.	DOWNLOADING DATA	
9.4.	SETTING TIME AND DATE	

1. PRELIMINARY REMARK

This technical information requires the knowledge of the sat801 Instructions for Use manual. It is assumed that the operation and menu structure of the sat 801 are known. In individual cases, it is necessary to use the Instructions for Use manual together with this technical information.

2. FUNCTION DESCRIPTION

2.1. Intended Use

Continuous, non-invasive monitoring of the functional oxygen saturation of arterial hemoglobin (SpO2) and the pulse rate (measured by the SpO2 sensor) in adults, children, and neonates for use in hospitals, home care, for mobile or stationary applications. Monitoring may only be carried out with approved Masimo sensors.

2.2. Contraindication

Disposable SpO2 sensors of the Masimo LNOP series shall not be used in patients who show an allergic reaction to the adhesive tape.

The sensors must be removed and repositioned every 8 hours and as soon as indicated by insufficient blood circulation or other skin irritations.

Reusable sensors of the Masimo LNOP series shall not be used for long-term applications. These sensors are not suitable for long-term monitoring. They must be removed and repositioned every 4 hours and as soon as indicated by insufficient blood circulation or other skin irritations.

2.3. Product description

The sat 801 pulse oximetry system comprises the sat 801 pulse oximeter, based on the Masimo SET technology, a plug-in power pack, a patient cable, and different disposable or reusable SpO2 sensors.

The sat 801 pulse oximeter consists of a housing that is suitable for stationary (e.g. hospital, sleep laboratories, home care) and, due to its small size and low weight, also for mobile use inside and outside of health care facilities such as emergency and home care applications.

The sat 801 has a customized LC display for indication of numerical SpO2 and pulse values and the status of the unit. Additional LED displays indicate the power supply and alarm status of the device. Vital signs, sensor, status, and system alarms are given visually and/or acoustically. Upper and lower alarm limits can be adjusted both for SpO2 and for pulse rate.

5

The unit is operated by means of four buttons.

The sat 801 is based on the Masimo MS-13 OEM board. Communication with the device control hardware takes place at 9600 baud via a serial interface.

The sat 801 has an internal memory of 2 Mbytes for every-second recording of at least 160 hours of SpO2, pulse rate and IQ monitoring. The internal memory can be read out via the USB interface only.

The sat 801 is powered via an external plug-in power pack with a wide-range voltage input.

2.4. Theory of operation

The function principle of the sat 801 pulse oximeter with Masimo-SET is based on the fact that oxyhemoglobin and desoxyhemoglobin differ in their capability of absorbing red and infrared light, that the volume of arterial blood in the tissues varies from pulse to pulse (thus also the light absorption by the blood) and the arteriovenous shunting is very variable and the fluctuating absorption by the venous blood is the dominant noise component.

Since oxyhemoglobin and desoxyhemoglobin absorption capabilities are different, the amount of red and infrared light absorbed by the blood is related to the oxygen saturation of the hemoglobin. Masimo-SET divides the red and infrared pulsating portion of the absorption signal into an arterial and a noise signal thereby calculating the ratio of the arterial signals without the noise portion. The ratio of the two arterial absorption values is used by an empirically derived equation in order to determine the oxygen saturation. The values in a table are based on studies conducted with patients whose oxygen saturation values measured with Masimo-SET technology were compared with the results obtained with an Co oximeter.

6

3. TECHNICAL DESCRIPTION OF THE SAT 801

4. ASSEMBLY

4.1. Displays and controls

4.1.1. Front

- 1 LCD-Display
- 2 Green latticed-LED
- 3 Yellow alarm LED
- 4 Red alarm LED
- 5 Backlight button and Down-arrow navigational button
- 6 Menu button
- 7 On-/Off button 😃
- 8 Alarm mute button
 and
 Up-arrow navigational button





4.1.2. Back

- 1 Loud speaker
- 2 Battery compartment -lock
- 3 Battery compartment





4.2. Connections on the unit

Connections on the upper panel of the unit:

- 1 USB-Interface
- 2 Patient cable connection
- 3 AC adapter socket



Fig.3: Upper panel of unit

4.2.1. Block diagram

The following block diagram describes the functional units of the sat 801.





All functions of the sat 801 are arranged on two printed circuit board:

- Masimo MS-13 board: pulse oximetry signal acquisition
- Main board: microprocessor with USB interface, buzzer , power supply and LCD

4.3. Power supply

The sat 801 is either powered via an external plug-in power pack (4.5 V) or by two AA batteries.

• Euro version: Egston N1EFM 12 W 4.5 V +/-1 %

The sat 801 works with two different internal operating voltages

- DC 5.0 volts: Masimo board
- DC 3.3 volts: Main voltage

A circuitry is available which protects the device against incorrect polarity and overvoltage. No sufficient voltage is supplied if the input voltage is less than 4.0 volts. The input voltage is measured at 0.125-second intervals. If this voltage is below the specified threshold, the device switches to battery mode. The battery voltage monitoring then applies.

The current battery voltage is measured at 0.125-second intervals, averaged over 10 seconds, and the resulting value is continually shown graphically on the display.

4.4. ON/OFF button

When the ON/OFF button is pressed, the sat 801 is waked from the sleep mode.

4.5. Masimo board

The Masimo OEM board and the CPU communicate via a 9600 baud serial interface.

4.6. Microprocessor and peripherals

A **microcontroller**, type ATMEL Atmega 329V, controls the task sequence of the sat 801. It belongs to the AVR series and has an 8-bit core, an internal in-System Self-Programmable 32 kByte flash EPROM, an internal 2 kByte static RAM, an internal 1 kByte EEPROM. In addition, the Atmega 329V has, amongst other things, an integrated 4 x 25 segment LCD driver, 10- bit A/D converter, programmable Serial USART, master/slave SPI serial interface, programmable watchdog timer with separate on-chip oscillator, an interrupt controller, a 2-channel 8-bit timer, a 16-bit timer, and a real time counter with separate oscillator.

A **real time clock module DS1374C** is used to keep a real time and as a additional external watchdog timer with a directly buzzer control possibility.

The loudspeaker is controlled via a timer output (pitch signal) and a pulse width output (volume signal). This solution relieves the microcontroller from the task of direct waveform generation and ensures a good tone quality. The loudspeaker can be tested for "short-circuit" and "no contact" error conditions. If such a system occurs, an additional buzzer will be activated.

4.7. Self-diagnostic capability of the sat 801

4.7.1. Watchdog and undervoltage reset

To check the watchdog (WDT) function, the WDT toggle signal is interrupted and the reset function is checked at undervoltage (voltage divider simulates undervoltage) after program start.

If these tests are not passed, respective system alarms are generated.

4.7.2. ROM check

The internal flash program memory is checked upon initialization and at regular time intervals (< 10 h) according to the CRC-16 method.

If the ROM check is not passed, a system alarm is generated.

4.7.3. RAM check

The whole internal RAM area is checked once before the major loop is started. If the RAM check is not passed, the sat 801 is rebooted.

4.7.4. Clock module check

The data in the NVRAM are protected with CRC-16. After each device start from the poweroff status, the data are loaded from the NVRAM and checked using the CRC- 16 method. If the CRC is not correct, a system alarm is generated.

4.7.5. Communication check with the Masimo board

After power-on and completion of the above mentioned internal self tests, the Masimo board is supplied with voltage, reset, and initialized.

All data packages received by the Masimo board are constantly checked for start and stop bytes as well as for checksum.

If no new data package is sent by the Masimo board within 2.5 seconds, a system alarm is triggered.

In addition, the board mode, the algorithm mode, the averaging time, and the set data format are checked for consistency once every second.

In the event of deviations, a medium priority alarm is generated.

If more than 5 transmission errors occur within 2 seconds, or if incorrect board mode, algorithm mode, averaging time, or data format information is detected more than 4 times, a system alarm is also triggered.

4.7.6. Operating voltage check

Mains voltage check

The battery voltage is measured at 0.5-second intervals and the measured value is averaged over a period of 10 seconds. If the calculated average falls below a predefined 1-hour alarm threshold, the 1-hour battery alarm (low priority) is generated. If the calculated average value falls below the 15-minute alarm threshold, the 15- minute battery alarm is generated. If the average value falls below the U_battery-flat threshold, a system alarm is generated. If the average value falls below the U_switch-off threshold, the device is switched off automatically.

Input voltage check

The input voltage is measured at 0.5-second intervals. If this voltage is below the specified threshold, the current operating mode is recognized as battery mode. The above mentioned battery voltage monitoring then applies.

4.7.7. Loudspeaker check

The loudspeaker is checked once after program start, and then each time before an audio alarm is generated. Two potential error conditions are detected: (short-circuit and no-contact). If the check is not passed (= loudspeaker is faulty), all audio signals are then issued via the buzzer. If the error has been detected, an additional low priority alarm is started.

4.7.8. Stack monitoring

The stack is continually monitored for overflow status. If the stack pointer exceeds a predefined limit, a system alarm is generated.

4.7.9. Detection of an uncontrolled reset

After the program start, the system always checks whether or not the previous monitoring was completed as specified. The monitoring status at the start of monitoring is stored in the NVRAM area of the clock module. If monitoring was completed as specified, the

12

respective status is deleted. If a respective monitoring status is detected after program start, the system recognizes this as an uncontrolled reset. A system alarm is triggered.

5. MAINTENANCE PROCEDURES

This chapter covers the following items:

• Regular maintenance procedures to be carried out on the sat 801

5.1. Preliminary remark

The following sat 801 components are subject to wear:

• Li battery

All other components are not subject to ageing.

5.2. Battery replacement

The manufacturer recommends replacing the Li battery at 10-year intervals.

Note !

Since the service philosophy is a complete-replacement strategy, the Lithium battery may only be changed in the manufacturer's service department.

6. POSSIBLE ERRORS AND THEIR CORRECTION

This chapter covers the following items:

- Description of possible errors
- Information about their causes and correction

6.1. Error: System alarms

6.1.1. Hardware-related system alarms

All hardware-related system alarms need to be serviced at the manufacturer.

6.2. Error: Green AC power LEDs do not come on

Error situation:	Green AC power LEDs do not come on
Error correction:	1. Check function of plug-in power supply
	2. Hardware failure: send unit in

6.3. Error: sat 801 cannot be switched on in battery mode

Error situation:	sat 801	cannot	be	switched	on i	n l	oattery	mode

Error correction: 1. Replacement of the batteries

2. Hardware failure: send unit in

6.4. Error: sat 801 will not switch on in mains mode

Error situation: sat 801 will not switch on in mains mode

Error correction: 1. Check function of plug-in power supply

2. Hardware failure: send unit in.

6.5. Error: Yellow and/or red alarm LEDs do not come on

Error situation:	Yellow and/or red alarm LEDs do not come on
Error situation	is confirmed by visual self test
Error correction:	Hardware failure: send unit in.

6.6. Error: Faulty loudspeaker

- Error situation: Loudspeaker is not activated, buzzer is activated instead of the loudspeaker
- Error correction: 1. Hardware failure: send unit in.

6.7. Error: Display values not plausible and/or faulty

- Error situation: The displayed values are not plausible or not within specification.
- Error correction: 1. Replacement of the sensor
 - 2. Replacement of the patient cable
 - 3. Hardware failure: send unit in

7. SAT 801 CHECK ROUTINE

This chapter covers the following items:

• Complete check routine for testing the sat 801 function

7.1. Preliminary remark

All checks of this complete check routine can also be carried out individually. The operability of the device, however, can only be assessed after completing the entire routine.

7.2. Power supply check

Action		Result		
•	Connection of plug-in power pack to	•	Green LED on plug-in power pack comes	
	public power supply		on.	
•	Connection of plug-in power pack to the	•	Green LED comes on.	
	sat 801			

7.3. Device self-test and brief check

Ac	Action		Result		
•	Switching the device on using the	٠	sat 801 performs several self-tests in		
	ON/OFF button		short succession:		
		•	Watchdog test - internal		
		•	Undervoltage test - internal		
		•	Reset test - internal		
		•	LC display test - visual check for pixel		
			errors		
		•	LED test. visual check to find out		
			whether or not both yellow and red LEDs		
			are on		
		•	Loudspeaker test. Acoustic check		

7.4. Alarm limits check

Note: The upper and lower alarm limits must differ by at least 2 units.

Action		Result		
•	Deletion of data memory	•	Existing data are deleted.	
•	Connection of a sensor for generation of	•	SpO2 and pulse values are displayed	
	physiological signals		after a short delay	
•	Adjusting the upper alarm limit for SpO2	•	"SpO2 high" alarm is triggered	
	to a value below the displayed SpO2			
	value			
•	If possible: Adjusting the lower alarm	•	SpO2 low" alarm is triggered	
	limit for SpO2 to a value above the			
	displayed SpO2 value			
•	Adjusting the upper alarm limit for pulse	•	"Pulse high" alarm is triggered	
	to a value below the displayed pulse			
	value			
•	Adjusting the lower alarm limit to a value	•	"Pulse low" alarm is triggered	
	above the displayed pulse value			
1		1		

7.5. Sensor alarms check

Action		Result		
•	Switching on the device while the sensor	•	sat 801 changes to search mode, then	
	is put on.		displays the current SpO2 and pulse rate	
			values.	
•	Disconnecting the sensor from the	•	Sensor alarm is triggered	
	patient connecter while the sat 801 is			
	operating			
•	Reconnect sensor to sat 801	•	Alarm stops	
•	Remove sensor from finger	•	Sensor alarm is triggered	
•	Reapply sensor on finger	•	Alarm stops	
•	Slightly lift sensor off finger and direct	•	Sensor alarm is triggered	
	light from a light source to sensor			

7.6. Acoustic signals check

Action		Result		
٠	Selection of alarm tone volume from 1 to	•	Alarm tone volume varies from very low	
	10		to very loud	
•	Selection of alarm tone OFF	•	Alarm tone is switched off. At the same	
			time, the symbol appears on the LC	
			display.	
•	Selection of pulse tone volume from 1 to	•	Pulse tone volume varies from very low	
	10		to very loud	
•	Selection of pulse tone OFF	•	Pulse tone is switched off	

7.7. Device validation

This test requires the use of the pulse oximetry simulator SmartSat from Clinical Dynamics and cannot be carried out on a patient. Additional test equipment: Stop watch.

These tests are not part of standard service.

Contact the manufacturer for a detailed test description.

7.8. Brief check

A brief check is carried out by allowing the device to complete its self-test.

1. Connect the sat 801 to the public power supply via the plug-in power pack.

2. Switch the sat 801 on.

3. Allow it to complete its self-test.

4. The brief check is completed successfully if the device reaches the monitoring mode without generating a system error.

5. Switch the sat 801 off.

8. SERVICE PROGRAM SERVICE801

This chapter covers the following items:

- Using the service program
- Performing software updates

8.1. Preliminary remarks

The service program allows you to read out, modify and save again device constants,

parameters and settings stored in the sat 801. In addition, it can be used to update the sat 801 firmware and the Masimo board software.

This requires a PC with a USB interface and an operating system W2k or later (free memory space: at least 10 Mbytes).

8.2. General information about the Service801

The welcome screen appears after starting the sat 801 service program:

🥃 sat801			
Alam limits Sp02 high: Sp02 low: Pulse high: Pulse low: Masimo Averaging time: Sensitivity:	Acoustics Pulse tone:	Mode Access code clinic: Access code home: Home-Clinic Clinic Mode Clinic Mode Data mode Owerwrite Freeze	set801 Parameters set801 S/N: Mainboard S/N: Masimo-Board S/N: SW/Version: Update
turn on the sat801			

In the left area you are able to

- display and modify alarm limits for SpO2 and Pulse Rate
- display and modify the SpO2 signal processing algorithm parameters such as averaging time and sensitivity
- display and modify the acoustical settings
- display and modify the access codes for clinical and home care mode

In the right area you are able to

- identify the serial numbers for the internal PCBs
- identify the firmware version
- perform a firmware update

8.3. Performing a parameter update

- 1. Setup the connection between the PC and the sat 801.
- 2. Start Service801. The welcome screen is displayed.
- 3. Turn on the sat 801.
- 4. All stored parameters are displayed.

2 set801 - Conversionalise								
Adambinats DpCD high 101 DpCD high 101 PAdar high 201 Robins Averaging these 111 Averaging these 111	Abadim Publice D d Manive D d Mage lan (D d)	Male darens ook tein: 000 darens ook teine 000 Faar Chin F Daal Hale F Daal Hale F Daal Hale F Daal Hale F Daal Hale	sell(1 Pasentes netII1154t 1990042 Mathoad54t 05144/95002 Manhodhaed54t 900000 Set Vesion 11.05					
serence, p	1 Tarani		Update					

- 5. Perform the necessary modifications in the masks.
- 6. Click on the **Transmit** button to allow for parameter update of the sat801.



- 7. Remove the USB cable from the sat 801.
- 8. The sat 801 automatically switches off.

a. Performing a software update

- 1. Setup the connection between the PC and the sat 801.
- 2. Start Service801. The welcome screen is displayed.
- 3. Turn on the sat 801.
- 4. All stored parameters are displayed.



5. Click on the **Update** button to allow for a software update of the sat801.



- 6. Remove the USB cable from the sat 801.
- 7. The sat 801 automatically switches off.

9. DATA DOWNLOAD PROGRAM DOWNLOAD801

This chapter covers the following items:

- Using the Download801 program
- Performing a data download

9.1. Preliminary remark

The Download801 program allows you to transmit stored monitoring data from the sat801 to a PC. This data can be further analysed by an oximetry program such as TrendLibrary.

9.2. General information about Download801

With Download801, you are able to

- set and synchronize the system time of the sat 801
- perform a data download procedure

9.3. Downloading data

- 1. Setup the connection between the PC and the sat 801.
- 2. Start Download801.
- 3. Turn on the sat 801.
- 4. Automatically, the current sat 801 time setting, S/N and software version are transmitted to the PC and displayed.



5. Click on **Transmit** to allow for the data download. The data download starts.



6. You will then be prompted for the target file.

Speichern unt	IF				? 🔀
Speichern in:	Eigene Dateie	m	• +	۵ 🖻 📼	
Zidetet vesvendete D Denktop	CyberUnk Eigene Bider Eigene Musk Eigene Musk Eigene Videce Meine Shapes				
Eigene Dateien					
Advelopidrz					
Netzverkungeb ung	Dateigane:	INCOMPANY AND A		•	Speichen

- 7. After successful storage of the file, remove the USB cable from the sat 801.
- 8. The sat 801 automatically switches off.

9.4. Setting time and date

- 1. Setup the connection between the PC and the sat 801.
- 2. Start Download801.
- 3. Turn on the sat 801.
- 4. Automatically, the current sat 801 time, S/N and software version is transmitted to the PC and displayed.

🧧 sat801 - 0	Communication		×
PC-time: sat601-time:	13.09.2006 09:17:34 13.09.2006 09:11:36 Transfer PC-time	oat901 S/N: 10950012 SW/Version: 1.06 Download data	
>			

5. Synchronize the PC-time with the sat 801 time by clicking **Transfer PC-time**.

🖬 sat801 - 0	Communication	
PC-time: sat801-time:	13.09.2006 09.27.25 13.09.2006 09.27.25	s#801 S/N: 10950012 SW/Version: 1.06
>	Transfer PC-tane	Uownload data

- 6. Remove the USB cable from the sat 801.
- 7. The sat 801 automatically switches off.